15mL DIAL-A-DOSE INJECTOR OR DRENCHER

The Phillips 15ml Automatic Injector or Drencher is a quality precision instrument, designed for mass treatment of livestock. Given normal care it will last indefinitely. As components in this instrument may be affected by solvents in some commonly used farm chemicals no responsibility will be accepted by the manufacturer should the instrument be used with such products.

BEFORE USE: Always read the label.

Check the label on the pharmaceutical manufacturer's container for dose rates, precautions, and safety information prior to use.

Use only the recommended dose rate.

Use only the pharmaceutical manufacturer's recommended rates. Refer to the pharmaceutical manufacturer's dose rate chart or specification. NJ Phillips Pty Limited will take no responsibility if the injector or drencher is used for any other purpose than specified or used contrary to the pharmaceutical manufacturer's dose rate specifications.

MAIN FEATURES MAKING THIS INSTRUMENT SO PRACTICAL EASY ACCESS - Figure 1

To access the internal parts, it is important to hold the instrument as the return spring will force the handle and lever apart under pressure. Now simply remove the handle clip (2) by sliding it out the groove in the handle. When the pushrod assembly comes free of the handle, access is then able to be made to the inlet valve spring assembly and to clean or replace the piston o-ring. When re-assembling, ensure the piston o-ring is lubricated with NJ Phillips Lubricant, then slide the pushrod assembly back into the handle, ensuring the return spring is located correctly in the handle and lever set. Then, secure the pushrod assembly by sliding the handle clip (2) back into the slot within the handle.

FIGURE 1: EASY ACCESS

FIGURE 2: INLET VALVE SPRING ASSEMBLY

FIGURE 3: DELIVERY VALVE SPRING ASSEMBLY

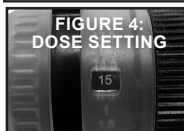
FIGURE 4: DOSE SETTING

SUPERIOR VALVE DESIGN - Figure 2 & 3

The inlet valve (3) is housed in the front of the push rod and is accessed by removing the inlet valve spring assembly by holding the spring and turning in an anticlockwise direction to pull out of the front of the piston. To reassemble once cleaned or being replaced, re-lubricate with NJ Phillips Lubricant, locate the inlet valve (3) of the assembly into the front of the piston and gently twist and push the valve spring assembly into the retaining clips.

The delivery valve is located at the rear of the needle mount or drench nozzle. The delivery valve may be readily removed for cleaning.

TO SET DOSE - Figure 4

The dose can be set from 0.5ml to 15ml in 0.5ml increments. The dose is very easily set. You simply turn the dose adjuster to the dose number you require in the visible window. Once the dose is set, squeeze the handle and lever and check the front of the piston is stopping at the required dose.

Ensure the handle and lever are in the fully open position before changing the dose. Do not squeeze to change the dose as this will damage the instrument.

VISIBLE DOSE

Transparent graduated cylinder provides instant check of dose.

NEEDLE MOUNT & NUT - INJECTOR

The injector is fitted with a metal needle mount and Luer lock nut. To fit the needle ensure the needle nut is screwed all the way on to the end of the cylinder. Locate your needle on to the mount and twist clockwise to lock it in place. To remove the needle, twist it in an anticlockwise direction.

DO NOT store your applicator or feed tube full of product. Clean as per the "After Use" instructions.

INSTRUCTIONS FOR USE

- 1 Connect the handpiece to the container of material by means of the connecting tube and squeeze the lever several times to prime the instrument.
- 2 The dose can be set from 0.5ml to 15ml in 0.5ml increments. The dose is very easily set. You simply turn the dose adjuster to the dose number you require in the visible window. Once the dose is set, squeeze the handle and lever and check the front of the piston is stopping at the required dose.
- 3 Check dosage accuracy. It is a wise practice to check the dose with each new container used. Use a measuring flask to test the dose.

PRIMING INSTRUMENT: As this is a dial-a-dose instrument, the dose must be set at maximum to prime the instrument. This must be repeated every time a new container of material is used.

Care must be taken to ensure the liquid does not come into contact with any part of the operators body. Chemicals may cause injury to the operator.

INJECTING

The injection attachment is designed for use with Luer needles only. It is most important that the chosen site be clean - ideally swab the skin with surgical or methylated spirit. Read the chemical manufacturers' label and literature on dose rates and site of injection.

The following points are worth remembering.

- 1 Sterilize the instrument and needles (use cold water sterilant only - DO NOT BOIL OR AUTOCLAVE)
- 2 Always inject into loose skin.
- 3 After injecting massage the site to disperse the injected solution.

AFTER USE

- 1 Thoroughly flush equipment with a mild warm water detergent mix.
- 2 Flush again with clean water until all detergent is removed.
- 3 Draw a small quantity of NJ Phillips Lubricant into the cylinder through the inlet fitting.

DRENCHING


Ensure that all equipment is thoroughly clean before use, by flushing with water. Before each use the nozzle should be inspected to ensure plating is not damaged or worn to a sharp edge. Should this occur, remove with file or emery paper or replace nozzle.

After Drenching: Where suspension type drenches have been used it is advisable to flush the complete equipment with a water detergent mix. This should be followed by clean water. Solution type drenches normally require only a thorough flushing with clean water.

All moving parts should be lubricated before storage.

LUER LOCK NEEDLE FITTING

1. Position needle in front of needle nut.
2. Locate needle onto needle mount. Turn clockwise to tighten.
3. Needle now secured.

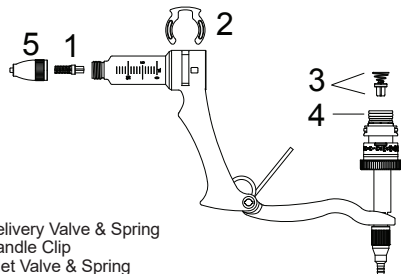1. Delivery Valve & Spring
2. Handle Clip
3. Inlet Valve & Spring
4. Piston O-Ring
5. Luer Needle Mount Assembly

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