
# 5 ML METAL TUBE FED INJECTOR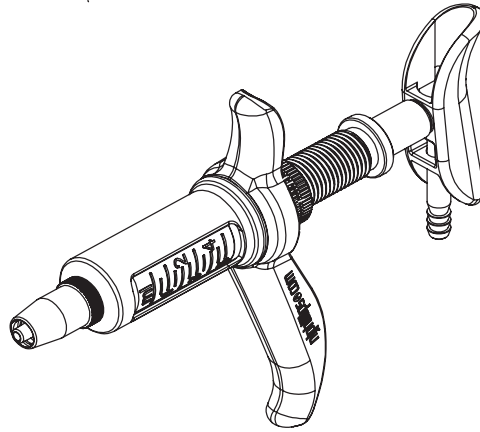

## ENGLISH

### INSTRUCTIONS FOR USE

#### Sterilization:

It is essential that this instrument, and a supply of needles, be thoroughly sterilized before each use. A common method of sterilization is as follows:

- 1 Attach connecting hose to handpiece.
- 2 Wrap cloth around handpiece and draw water into the cylinder through the connecting hose.
- 3 Suspend the complete instrument in a container of water and boil, together with the needles, for 10 to 20 minutes.
- 4 Remove instrument from container, wrap cloth around the handle and pump dry.

Suspending the instrument not only makes it easier to remove, but also prevents damage should the container boil dry. Chemical sterilization with antiseptic solutions is sometimes practised and in such instances the recommendations of the chemical manufacturer should be followed. **DO NOT attempt to sterilize by autoclaving.** It is neither necessary, nor desirable, to dismantle the handle, or any other component, before sterilizing.

#### To set the dose:

- 1 Loosen dose adjuster lock nut (14).
- 2 Rotate threaded dose adjuster (15) until the front piston washer registers with the required calibration on the cylinder.
- 3 Retighten dose adjuster lock nut (14). After attachment to the vaccine container hold the instrument vertically (needle upwards) and prime until all air is removed from the cylinder and connecting hose. Presence of excessive air bubbles in the cylinder indicates a loose fitting or damaged piston seal. See instructions for care and maintenance for replacement procedure.

**Care must be taken to ensure the liquid does not come into contact with any part of the operators body. Chemicals may cause injury to the operator.**

#### CARE AND MAINTENANCE AFTER USE

- 1 Remove all traces of material from instrument by flushing thoroughly with a warm water detergent mix. Follow with clean, warm water.

- 2 Disconnect feed tube from handpiece and place inlet fitting in NJ Phillips Lubricant, allowing a small quantity of NJ Phillips Lubricant to be pumped through the handpiece.

- 3 Store with dose set at 1 ml. This is most important as, after being stored for some time, the piston washers may adhere to the cylinder. Setting down to 1 ml does not allow sufficient stroke to damage the washers when the instrument is worked for next use.

**DO NOT store your applicator or feed tube full of product. Clean as per the "Care and Maintenance" instructions.**

#### Lubricating Instructions:

If this instrument becomes sluggish in operation, lubrication is probably required behind the piston.

- 1 Unscrew the cylinder lock nut (13) until it can be pulled back against the spring and clear of the body.
- 2 Lubricate the cylinder with four or five drops of NJ Phillips Lubricant.
- 3 Screw back the cylinder lock nut (13) to the required dose.

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