njphillips™

DATAMARS

Assembled at an ISO 9001:2015 accredited facility
Contact your local NJ Phillips product
representative for service kit details
Email: ahdsinfo@datamars.com
Website: njphillips.com
Toll free number: 1800 247 175 (Australia)

355 0000-740 QL620A

English

GENERAL ADVICE FOR USE OF NJ PHILLIPS INJECTORS & INJECTOR DRENCHERS

PLEASE READ THIS LEAFLET BEFORE USE. KEEP THIS LEAFLET AND
OTHER INSTRUCTIONS IN A SAFE PLACE FOR FUTURE REFERENCE.

WARNINGS

1. THIS APPLICATOR IS DESIGNED TO INJECT ANIMALS. THE OPERATOR MUST TAKE CARE TO AVOID CONTAMINATING THEMSELVES WITH THE PRODUCT BEING USED OR CAUSING THEMSELVES NEEDLE-STICK INJURIES OR SELF-INJECTION.
2. IT IS ESSENTIAL THAT THE OPERATOR READS AND FOLLOWS THE DIRECTIONS FOR USE AND SAFETY INSTRUCTIONS ON THE LABEL OF THE PRODUCT BEING APPLIED. THE OPERATOR IS AT RISK OF INJURING THEMSELVES, OTHER PEOPLE, OR THE ANIMALS BEING TREATED IF THE PRODUCT THEY ARE APPLYING IS NOT HANDLED SAFELY AND USED CORRECTLY.
3. THE OWNER OF THIS APPLICATOR MUST ENSURE THAT THE OPERATOR IS TRAINED IN ITS CORRECT USE. THE OPERATOR IS AT RISK OF INJURING THEMSELVES, OTHER PEOPLE, OR THE ANIMALS BEING TREATED IF THE APPLICATOR IS NOT USED CORRECTLY.
4. WHEN FITTING OR REMOVING NEEDLES AND DURING USE, DUE CARE AND ATTENTION MUST BE GIVEN TO ENSURE THAT ACCIDENTAL NEEDLE STICK INJURIES ARE AVOIDED.
5. WHEN SETTING UP CLEANING, MAINTAINING OR USING THIS APPLICATOR, TAKE ALL APPROPRIATE PRECAUTIONS TO AVOID SPILLING, DIRECT CONTACT WITH, OR INHALATION OF, THE PRODUCT BEING USED WITH THIS APPLICATOR. REFER TO THE USE AND SAFETY INSTRUCTIONS ON THE LABEL OF THE PRODUCT FOR SPECIFIC ADVICE AND WEAR APPROPRIATE SAFETY CLOTHING AS RECOMMENDED BY THE MANUFACTURER OF THE PRODUCT BEING APPLIED.
6. KEEP PACKAGING MATERIAL AWAY FROM CHILDREN AND ANIMALS. PLASTIC BAGS AND PACKAGING CAN CAUSE SUFFOCATION. PLEASE DISPOSE OF ALL PACKAGING CORRECTLY.
7. BEFORE USE THE EQUIPMENT MUST BE STERILISED.
8. SEEK ADVICE ON THE STANDARD PRACTICES TO BE USED TO MINIMISE THE RISKS OF CROSS-INFECTION BETWEEN TREATED ANIMALS. SUCH ADVICE SHOULD BE AVAILABLE FROM SOURCES SUCH AS PRODUCER AND INDUSTRY ORGANISATIONS, GOVERNMENT EXTENSION SERVICES OR THE MANUFACTURER OF THE PRODUCT BEING APPLIED.
9. BEFORE DOSING, ENSURE THAT THE COMPONENTS OF THE APPLICATOR ARE SECURE AND THAT IT IS IN GOOD WORKING ORDER.
10. DO NOT USE THIS APPLICATOR IF IT IS LEAKING. STOP IMMEDIATELY AND CORRECT THE FAULT BEFORE

CONTINUING.

11. THE DOSE ADJUSTMENT MECHANISM OF THIS APPLICATOR MAY BE DISLODGED DURING HANDLING. REGULARLY CHECK THAT THE DOSE SETTING IS CORRECT.
12. DO NOT LEAVE THE EQUIPMENT EXPOSED TO DIRECT SUNLIGHT OR EXTREMES OF HIGH OR LOW TEMPERATURES AS THE APPLICATOR OR THE PRODUCT BEING APPLIED MAY BE AFFECTED.
13. DURING THE WORKING DAY, WHEN THE APPLICATOR IS NOT IN USE, LEAVE IT IN A SAFE PLACE OUT OF REACH OF CHILDREN, ANIMALS AND PEOPLE NOT TRAINED IN ITS CORRECT USE.
14. DURING THE WORKING DAY, WHEN THE APPLICATOR IS NOT IN USE, AND IF THE APPLICATOR IS OF A TYPE WHICH IS CONNECTED BY A FEED TUBE TO THE CONTAINER, ENSURE THE APPLICATOR IS ABOVE THE CONTAINER. IF IT HANGS BELOW THE CONTAINER FOR LONG PERIODS IT MAY DRIP.
15. FOLLOW THE SEPARATE CLEANING AND STORAGE INSTRUCTIONS SHOWN BELOW.
16. STORE THE APPLICATOR IN A SAFE PLACE OUT OF THE REACH OF CHILDREN. STORE NEEDLES SEPARATELY FROM THE APPLICATOR.
17. ALWAYS WASH HANDS THOROUGHLY AFTER USING OR SERVICING THIS EQUIPMENT.
STERILISING EQUIPMENT BEFORE USE

The recommendations for sterilisation vary according to the material from which the injector is made. Specific sterilisation instructions should be found on the packaging or leaflet accompanying this injector. **CLEANING & STORAGE**

DO NOT store your applicator or feed tube full of product. To ensure long service from this applicator it should be properly cleaned and stored after use:
1. After use, empty the remaining product and dispose of it appropriately.
2. Flush through with warm, soapy water, drawing through the applicator feed tube or bottle attachment.
3. Then flush through with clean water. Expel any excess water.
4. Before storing, disconnect the feed tube and inject a small quantity of NJ Phillips Lubricant into the inlet. Operate the gun several times until the lubricant appears in the barrel and the seals are lubricated.
5. Dry the outside surfaces of the applicator and store in a plastic bag in a safe place out of the reach of children.

YOU CAN CONTACT NJ PHILLIPS, AND OBTAIN MORE INFORMATION ABOUT THIS EQUIPMENT AND SPARE PARTS BY GOING TO WWW.NJPHILLIPS.COM

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